

D2 APPLIANCE

Simplifying Class II Correction

The D2 distalizing appliance was developed to simplify treatment of Class II cases during the initial stage of treatment. The patented D2 design treats to a Class I platform by the intra oral elastic forces. Molars are distally rotated into their desired position. Take advantage of early treatment patient cooperation.



Stainless Steel

Features & Benefits

- One size fits all reducing product inventory
- Not attached to upper / lower wires, not blocking sliding mechanics, brackets can be effective at the same time, optimizing
- Very low profile, avoiding contact lip/cheek tissue, no "frog mouth", all allowing for best patient comfort / compliance
- Easy to place, best efficiency
- Lower price than most classic telescoping systems and other similar systems
- Forces by elastics, no bulky telescoping / spring mechanisms (frog mouth), optimal hygiene, no food traps, no obstruction in *functioning corrector, greater efficiency.

4 EASY STEPS

Etch and prep teeth to be bonded. Bond Posterior Component



Mark & cut wire bar to determined length



Slide anterior component onto wire bar & bond to tooth



Bond lower molar and apply elastic forces.

